


USER INSTRUCTIONS

HALF MASK/FULL FACE MASK TEST ADAPTOR

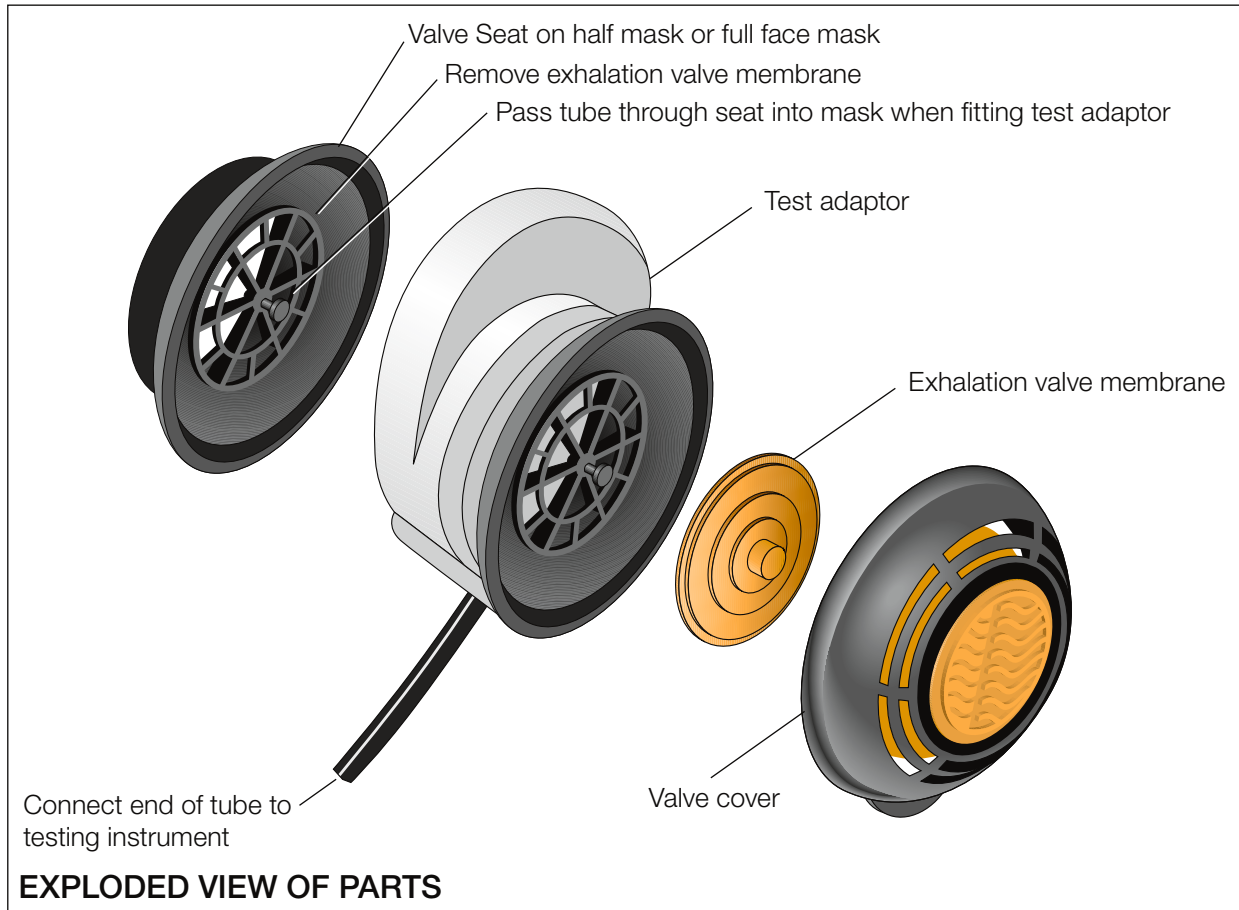

The SR 328 Test Adaptor is used to connect the half mask or full face mask to a measuring instrument for facial fit testing. The adaptor is fitted to an exhalation valve of the half mask or full face mask.

Procedure:

1. Remove cover on one exhalation valve;
2. Remove the valve membrane and inspect it to ensure it is clean and in good condition;
3. Fit the adaptor to the valve seat so that the hose passes through the valve and into the mask. The outer end of the hose should point downwards;
4. Check that the internal hose does not contact the user's face when wearing the mask.

The adaptor hose can now be connected to the testing instrument.

WARNING: This product is intended for use only for facial fit testing under test conditions. Not for use under real working conditions.

Sundström

Sundström Safety AB

Västergatan 4
SE-341 50 Lagan
SWEDEN

Phone: +46 10 484 87 00
E-mail: info@srsafety.se

Manufactured by The S.E.A. Group

North Shore Business Park
35/1 Jubilee Avenue
Warriewood NSW 2102
AUSTRALIA

Phone: +61 (02) 9910 7500
Fax: +61 (02) 9979 5364
TollFree: 1800-655 129
Email: sea.australia@theseagroup.com

www.srsafety.com